



WATERBORNE™, INC.

Clinical & Environmental Parasitology Products ♦ 6045 Hurst Street, New Orleans, LA, USA 70118-6129
Tel. & FAX: 504-895-3338 or Toll-Free: 866-895-3338 ♦ Email: custserv@waterborneinc.com ♦ Website: www.waterborneinc.com

C101. BlockOut™ Counterstain

This product consists of a solution of Evans Blue dye in PBS with 0.04% sodium azide as preservative. The volume is 3.5 mL. It is conveniently packaged in a plastic dropper vial. This product can be used in up to 75 tests, using one drop per test (at approximately 45 uL per drop).

BlockOut™ Counterstain binds nonspecifically, fluorescing red using a fluorescein filter setting, thereby providing and enhancing contrast with cells that are fluorescing green due to binding of fluorescein. Generally, an incubation period of 1 minute with counterstain, after the antibody incubation step, gives good results.